
RECHARGEABLE
Ultra Ear™ BTE
HEARING ENHANCER

USER MANUAL

BATTERY CHARGING:

1. Insert USB charging cable to an AC charger (not included) or a computer USB port.
2. Insert the barrel plug end of the USB cable into the Hearing Enhancer's charge port.
3. A green LED indicator lights up while the unit is being charged. Once battery is fully charged, the light will turn off.
4. Battery charging time is approximately 2 hours.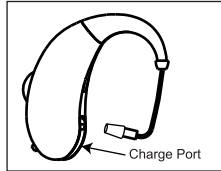

FITTING INSTRUCTIONS:

1. Place the Ultra Ear™ behind either ear.
2. Turn the ear tip to the appropriate position for left or right ear.
3. Insert ear tip into ear. Ear tip must be inserted properly into the ear.


The Ultra Ear™ contains three different size ear tips for the best possible fit.

OPERATING INSTRUCTIONS:


1. Turn the volume control to low level.
2. Place the hearing ear tip into your ear – left or right.
3. Increase volume control to a suitable level.

(Don't use your hand to cover the unit or it will make a high-pitch noise.)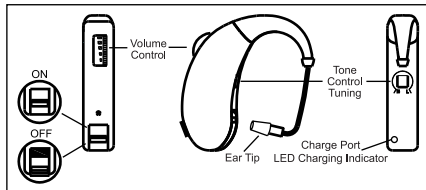


STONE CONTROL TUNING:

Use the plastic stick end to turn the Tone Control Tuning to adjust as "L" Low & "H" High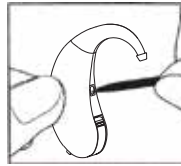

MAINTENANCE WITH ACCESSORIES:

Use the plastic piece with steel tip to pass through the ear tip end to remove earwax. Use the soft brush to gently brush across the ear tip to remove dirt & earwax.

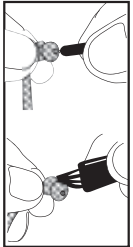CAUTION:

1. This product is not waterproof; therefore the user must take care to prevent water from entering the unit.
2. It is important to increase the volume control gradually to avoid sudden increase in sound.
3. This product is not a medical device.
4. Not for use with firearms.
5. This is not a hearing protection device.

BATTERY LIFE TIME	48 WORKING HOURS
OPERATING CURRENT	1.0 ~ 2.0 mA
OPERATING VOLTAGE RANGE	2.7 ~ 4.2 V
POWER CONSUMPTION	8.4 mW
FULL-ON GAIN dB(90dB input)	> 115dB
SIGNAL TO NOISE RATIO	> 35 dB
FREQUENCY RESPONSE	100-15 K +/- 10 dB
HARMONIC DISTORTION	<5%
CHARGING TIME	2 HOURS
WEIGHT (BATTERY INCLUDED)	6.5 g
BATTERY VOLTAGE	3.70 V
BATTTERY CAPACITY	45 mAh

GSM, LLC
 P.O. Box 535189
 Grand Prairie, TX 75053
 www.walkersgameear.com
 Toll Free: 877-269-8490

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